Professional Veterinary Diagnostic Test
BioPronix product line
 For Veterinary use only
 For in vitro use only
 Manufactured by Agrolabo S.p.A.

INTERPRETATION OF THE RESULTS

Negative Result

The test is negative if a red line appears in window C (control line).

Positive Result

The test is positive if two red lines appear. One line will appear in window T (test line) and one in window C (control line).

Invalid Result

The test is invalid if no coloured band appears in window C, even if a coloured band appears in window T.

QUICK GUIDE

1. Using the provided pipette, add 1 drop of sample (whole blood, serum or plasma) into S window.


2. Add 2 drops of diluent to window S of the device.

3. Wait 10 minutes to read the results, not later than 15 minutes.


4. Interpretation of the results.

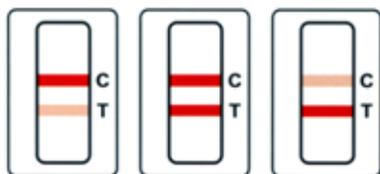Positive

Negative

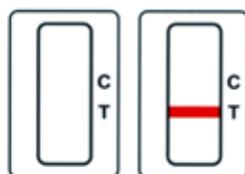Invalid

WARNING

Open the test package immediately before use. Place the device horizontally for testing. Keep the pipette in a vertical position when adding the sample to the device.

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HEARTWORM IC - Rapid immunochromatographic test for the detection of *Dirofilaria immitis* in canine whole blood, serum or plasma

TEST PRINCIPLE

Heartworm IC is a qualitative test that employs the immunochromatographic sandwich technique using monoclonal antibodies against *Dirofilaria immitis* antigen. A monoclonal antibody is conjugated with colloidal gold and a further monoclonal antibody is immobilized on the test membrane. If the sample contains *Dirofilaria immitis* antigens (positive sample), these antigens will bind to the colloidal gold conjugated antibody forming the Ab-Ag complex. The complex migrates along the membrane and will be captured by the second monoclonal antibody immobilized on the membrane at T window level, forming a red line (test line). The absence of the red line in the T window will indicate a negative result. The sample continues to migrate along the membrane up to window C, where a reagent able to bind the antibody conjugated with colloidal gold is immobilized, resulting in the formation of a red line (control line). The presence of the red line in window C confirms that the test was performed properly and should always appear, with both positive and negative samples.

KIT COMPONENTS

- Disposable pipettes
- Vial with diluent
- HEARTWORM IC devices

STORAGE AND SHELF LIFE

Store at room temperature (+18-25°C). When properly stored, the product remains stable for its full shelf life. Keep the kit away from excessive heat/cold or direct sunlight.

PRECAUTIONS

IMPORTANT: open individual device packaging immediately prior to use.
 For veterinary use only. Do not use kit components after expiry date.
 The test device should be placed horizontally on a flat surface while testing.
 Use a new disposable pipette for each sample.

SAMPLES

All samples need to be brought to room temperature before use. Previously frozen or stored samples must be centrifuged before use.
 Whole blood samples with anticoagulant (EDTA, heparin, etc) may be used. Whole blood samples should be fresh or stored in a refrigerator for no longer than 5 days.
 Serum or plasma that is fresh, or that has been refrigerated (+2-8°C) for no longer than 5 days, can be used. For longer storage, samples should be frozen.
 Haemolyzed samples may affect test results.

TEST PROCEDURE

1. Remove the test device from the protective pouch and place it on a flat surface.
2. Gently mix the sample by inverting. Holding the pipette approximately 1 cm from the device, add 1 drop of serum, plasma or whole blood sample into window S. Wait for the sample to be absorbed.
3. Holding the diluent bottle vertically, promptly add 2 drops of the diluent buffer to window S.
4. Read the result after 10 minutes and no later than 15 minutes.