Professional Veterinary Diagnostic Test
BioPronix product line
 For Veterinary use only
 For in vitro use only
 Manufactured by Agrolabo S.p.A.

INTERPRETATION OF THE RESULTS

Negative Result

The test is negative if a red line appears in window C (control line).

Positive Result

The test is positive if two red lines appear. One line will appear in window T (test line) and one in window C (control line).

Invalid Result

The test is invalid if no coloured band appears in window C, even if a coloured band appears in window T.

QUICK GUIDE

1. Remove the device from its wrapping and place it on a flat surface. Use the pipette to add 10 µl of serum, plasma or whole blood with anticoagulant into window S of the device.

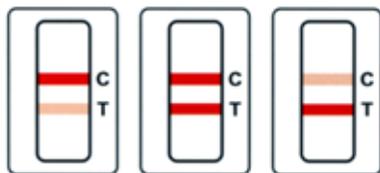
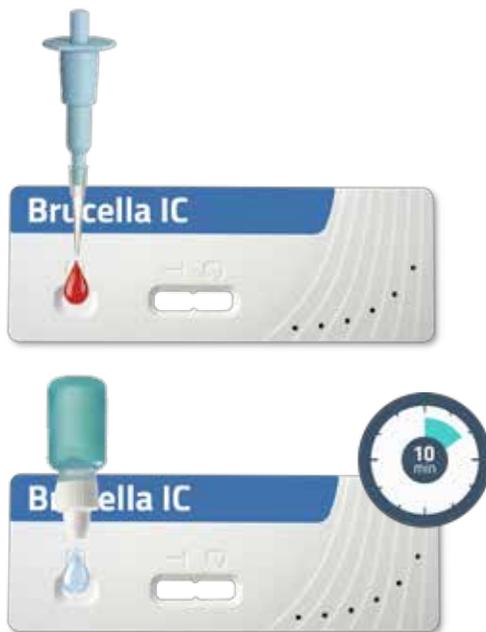
2. Hold the diluent bottle in a vertical position and add 2 drops of diluent in window S of the device.

3. Wait 5-10 minutes to read the results, not later than 15 minutes.

4. Interpretation of the results.

WARNING


Open the test package immediately before use. Place the device horizontally for testing. Keep the pipette in a vertical position when adding the sample to the device.

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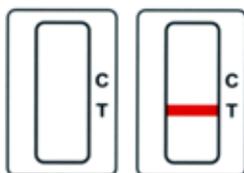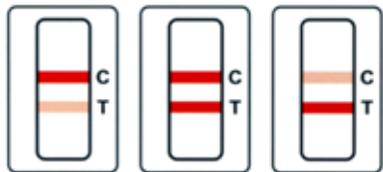
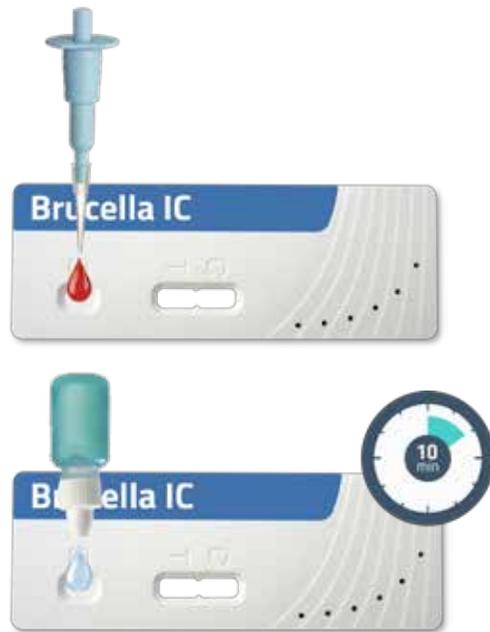
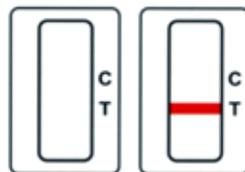
Negative

Invalid

BRUCELLA IC - Rapid test for the detection of antibodies to Brucella canis

USER MANUAL

Rev. R1 – 17/02/21

TEST PRINCIPLE

Brucella IC is an immunochromatographic test for the detection of anti-Brucella canis antibodies in canine whole blood, serum or plasma.

A specific Brucella canis antigen (Ag) is bound on the nitrocellulose membrane at the level of window T, while at window C level is immobilized a control reagent.

If the sample contains anti-Brucella canis antibodies (positive sample), these antibodies will bind to the colloidal gold conjugated Brucella antigen included in the pad of the test, forming the Ag-Ab complex. The complex Ag-Ab migrates along the membrane and will bind again to the Brucella antigen at T window level forming a red line (test line).

The sample continues to migrate along the membrane up to window C, where a reagent able to bind the antigen conjugated with colloidal gold is immobilized resulting in the formation of a red line (control line).

KIT COMPONENTS

- 10 µl Micropipette
- Tips
- Vial with diluent
- BRUCELLA IC devices

STORAGE AND SHELF LIFE

Store at room temperature (+18-25°C).

When properly stored, the product remains stable for its full shelf life.

Keep the kit away from excessive heat/cold or direct sunlight.

PRECAUTIONS

IMPORTANT: open individual device packaging immediately prior to use. For veterinary use only.

Do not use kit components after expiry date.

The test device should be placed horizontally on a flat surface while testing.

Use a new tip for each sample.

SAMPLE

All samples need to be brought to room temperature before use. Previously frozen or stored samples must be centrifuged before use. Whole blood samples with anticoagulant (EDTA, heparin, etc) may be used.

Whole blood samples should be fresh or stored in a refrigerator for no longer than 5 days.

Serum or plasma that is fresh, or that has been refrigerated (+2-8°C) for no longer than 5 days, can be used. For longer storage, samples should be frozen.

Haemolyzed samples may affect test results.

TEST PROCEDURE

1. Remove the device from its individual wrapping and place it on a flat surface.

2. Holding the micropipette add 10 µl of whole blood with anticoagulant, serum or plasma into window S of the device.

3. Holding the diluent bottle in a vertical position add 2 drops into window S of the device.

4. Read the result after 5-10 minutes and no later than 15 minutes.