AGROLABO
EhrlichiaCHECK Ab ELISA

EhrlichiaCheck Ab ELISA

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EhrlichiaCHECK Ab ELISA

Kit ELISA for the detection of IgG antibodies anti-Ehrlichia canis in canine serum or plasma samples

TEST PRINCIPLE

The test is performed as an indirect enzyme linked immunosorbent assay (indirect ELISA).

The antigen of Ehrlichia canis is fixed on the wells of the plate. The samples to be analyzed are distributed in each well. If the sample contains specific antibodies anti-Ehrlichia, these antibodies will bind to the antigen adsorbed, forming the Ag-Ab complex. After washing to eliminate all non-fixed material, the presence of anti-Ehrlichia antibodies is detected by a specific anti-canine IgG monoclonal antibody labeled with horseradish peroxidase (conjugate). All non-fixed material is removed by a second washing. A specific enzyme substrate is then added to the wells which causes a colorimetric reaction when the specific antibodies are present.

TEST COMPONENTS

- 1 x 96 wells plate (in strip of 8 breakable wells)
- 1 dropper of positive control - ready for use
- 1 dropper of negative control - ready for use
- 1 dropper of conjugate - ready for use
- 1 bottle of washing solution - 10X
- 1 bottle of diluent - ready for use
- 1 dropper of substrate - ready for use
- 1 dropper of stop solution - ready for use

MATERIALS REQUIRED BUT NOT SUPPLIED

Test tubes for serum dilutions; precision pipettes and tips; spectrophotometer at a wavelength of 450 nm.

PRECAUTIONS

1. Read the instructions carefully
2. Bring all reagents to room temperature (+20-25°C) prior to use
3. Do not substitute or mix instructions or reagents from different kits or lots
4. The substrate is very sensitive to the light and to contaminations. Do not use the tip directly in the vial. The stop solution is a diluted strong acid. Handle the two reagents with care
5. Do not use components after expiry date
6. Do not smoke, eat or drink where specimens or kit reagents are being handled
7. In each utilization of the kit the positive and negative control must be tested
8. Use a new tip for each sample
9. Observe each incubation time
10. Prepare each sample according to instructions
11. Avoid any contamination of the reagents
12. All kit components must be stored at +2-8°C

WASHING STEPS

The washing steps may be carried out using an automated plate washer or a multi-channel pipetting device suitable for dispensing 300 µl in each well. After the incubation, the washing steps should be carried out according to the following instructions:

- throw out the content of the plate by vigorously turning the plate over to avoid the possible mixture of the content from one well to another
- dispense a volume of 300 µl of wash buffer into each well
- shake the plate gently, avoiding contamination between wells
- turn over the plate and tap the wells vigorously to remove the wash buffer
- repeat the process as many times as is indicated by the kit's instructions
- prior to emptying out the plate's contents during the final washing step, ensure that the next reagent to be added to the plate is ready to use
- the wells should not be left dry for any longer than strictly necessary in order to dispense the reagent of the following step
- after the last washing step, blot the empty wells face down on clean paper towels or absorbent paper to remove residual wash buffer

PREPARATION OF SAMPLES

Canine serum or plasma samples.

Prepare a **1:100** dilution of each sample to be tested using diluent.

PREPARATION OF REAGENTS

Wash buffer solution:

Dilute one part of the wash buffer supplied into 9 parts of distilled or deionized water (ie. 100 ml of concentrated wash buffer with 900 ml of water). Once diluted, the wash buffer solution must be stored at +2 8°C.

TEST PROCEDURE

1. Bring all reagents to room temperature before use.
2. Add 100 µl of positive and negative controls in two wells. Add 100 µl of each dilution of the sera on the remainder wells of the plate.
 - **Delicately shake the plate, seal and incubate at room temperature for 15 minutes.**
3. Wash cycle 4 times according to the procedure previously described.
4. Dispense 100 µl of conjugate into each microwell.
 - **Delicately shake the plate, seal and incubate at room temperature for 15 minutes.**
5. Repeat the wash cycle 4 times according to the procedure previously described.
6. Add 100 µl of substrate to each well.
 - **Seal the plate and incubate at room temperature for 5 minutes in the dark.**
7. Proceed with visual or spectrophotometric reading as explained in points A and B:

- **A - Visual reading:** at the end of the incubation with substrate, **DO NOT ADD** stop solution.
- **B - Spectrophotometric reading:** at the end of the incubation with substrate, **ADD** 100 µl of stop solution to each well.

READING AND INTERPRETATION OF THE RESULTS

A - VISUAL READING – Without adding stop solution

At the end of the incubation with substrate, read visually, evaluating the coloration in the wells. The positive control must be blue/light blue and clearly distinguishable from the negative control, which should be colourless.

Interpretation of results

Positive samples: a blue colour will develop that indicates the presence of IgG anti-EHRlichia antibodies in the sample.

Negative samples: no colour will develop.

B - SPECTROPHOTOMETRIC READING - Add stop solution

The reading of the absorbance values (OD) must be carried out at a wavelength of 450 nm. If you run the samples or controls in duplicate wells, the OD values for the samples or controls will be the mean of both wells.

Test Validation

The test is to be considered valid when:

OD negative control < 0.1

OD positive control > 0.6

CUT-OFF value determination

Positive Cut-off = OD negative control + 0.28

Negative Cut-off = OD negative control + 0.21

Interpretation of results

SCREENING

Positive samples: samples with an OD higher than the positive cut-off value.

Negative samples: samples with an OD lower than the negative cut-off value.

Doubtful samples: samples with an OD between both cut-offs. In these cases, it is suggested to repeat the test after 3-4 weeks.

AGROLABO
EhrlichiaCHECK Ab ELISA

Manufactured by;
Agrolabo SpA

Head Office, Laboratories
and Production Centre
Diagnostic Division
Via Masero 59
10100 Scarmagno (TO)
Italy
Tel +39 0125 731111
Fax +39 0125 731190
E-mail agrolabo@agrolabo.it
www.agrolabo.it
shop.agrolabo.it