Professional Veterinary Diagnostic Test  
**BioPronix** product line  
 For Veterinary use only  
 For in vitro use only  
 Manufactured by Agrolabo S.p.A.

## INTERPRETATION OF THE RESULTS

### Negative Result

The test is negative if a red line appears in window C (control line).

### Positive Result

The test is positive if two red lines appear. One line will appear in window T (test line) and one in window C (control line).

### Invalid Result

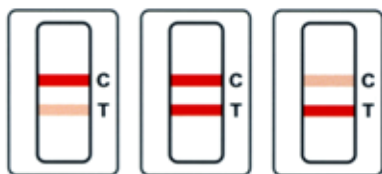
The test is invalid if no coloured band appears in window C, even if a coloured band appears in window T.

## QUICK GUIDE

1. Remove the device from its wrapping and place it on a flat surface. Use the pipette to add 1 drop of serum, plasma or whole blood with anticoagulant into window S of the device.
2. Hold the diluent bottle in a vertical position and add 3 drops of diluent in window S of the device
3. Wait 10 minutes to read the results, not later than 15 minutes.
4. Interpretation of the results.

## WARNING


Open the test package immediately before use. Place the device horizontally for testing. Keep the pipette in a vertical position when adding the sample to the device.

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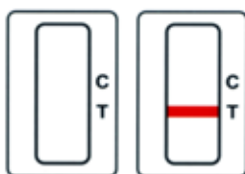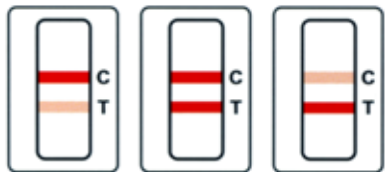
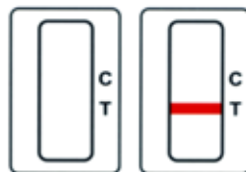
Negative

Invalid

## FCoV IC - Rapid immunochromatographic test for the detection of antibodies to Feline Coronavirus

## USER MANUAL

Rev. R1 – 25/06/21

### TEST PRINCIPLE

FCoV IC is an immunochromatographic test for the detection of anti-Feline Coronavirus antibodies in feline whole blood, serum or plasma.

If the sample contains anti-FCoV antibodies (positive sample), these antibodies will bind to the colloidal gold conjugated FCoV antigen included in the pad of the test, forming the Ag-Ab complex. The complex Ag-Ab migrates along the membrane and will bind again to the FCoV antigen at T window level forming a red line (test line).

The sample continues to migrate along the membrane up to window C, where a reagent able to bind the antigen conjugated with colloidal gold is immobilized resulting in the formation of a red line (control line).

### KIT COMPONENTS

- Disposable pipette
- Vial with diluent
- FCoV IC devices

### STORAGE AND SHELF LIFE

Store at room temperature (+18-25°C)

When properly stored, the product remains stable for its full shelf life.

Keep the kit away from excessive heat/cold or direct sunlight.

### PRECAUTIONS

IMPORTANT: open individual device packaging immediately prior to use. For veterinary use only.

Do not use kit components after expiry date. The test device should be placed horizontally on a flat surface while testing.

Use a new disposable pipette for each sample.

### SAMPLES

All samples need to be brought to room temperature before use. Previously frozen or stored samples must be centrifuged before use.

Feline whole blood samples with anticoagulant (EDTA, heparin, etc) may be used. Whole blood samples should be fresh or stored in a refrigerator for no longer than 5 days.

Serum or plasma that is fresh, or that has been refrigerated (+2-8°C) for no longer than 5 days, can be used. For longer storage, samples should be frozen.

Haemolyzed samples may affect test results.

### TEST PROCEDURE

1. Remove the device from its individual wrapping and place it on a flat surface.
2. Holding the pipette add 1 drop of whole blood with anticoagulant, serum or plasma into window S of the device.
3. Holding the diluent bottle in a vertical position add 3 drops into window S of the device.
4. Read the result after 10 minutes and no later than 15 minutes.