AGROLABO
FeLV ANTI-gp70 ELISA

FeLV anti-gp70 Ab ELISA

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FeLV anti-gp70 Ab ELISA

Kit ELISA for the detection of specific antibodies to anti-gp70 protein of Feline Leukemia Virus in feline serum or plasma sample

TEST PRINCIPLE

The test is performed as an indirect enzyme linked immunosorbent assay (indirect ELISA). The specific antigen of FeLV (gp70 protein) is fixed on the wells of the plate. The samples to be analyzed are distributed in each well. If the sample contains specific antibodies anti-gp70 FeLV, these antibodies will bind to the antigen adsorbed, forming the Ag-Ab complex. After washing to eliminate all non-fixed material, the presence of anti-gp70 FeLV antibodies is detected by a specific anti-feline IgG monoclonal antibody labelled with horseradish peroxidase (conjugate). All non-fixed material is removed by a second washing. A specific enzyme substrate is then added to the wells which causes a colorimetric reaction.

TEST COMPONENTS

- 1 x 96 wells plate (in strip of 8 breakable wells)
- 1 dropper of positive control - ready for use
- 1 dropper of negative control - ready for use
- 1 dropper of conjugate - ready for use
- 1 bottle of washing solution - 10X
- 1 bottle of diluent - ready for use
- 1 dropper of substrate - ready for use
- 1 dropper of stop solution - ready for use

MATERIALS REQUIRED BUT NOT SUPPLIED

Test tubes for serum dilutions; precision pipettes and tips; spectrophotometer at a wavelength of 450 nm.

PRECAUTIONS

1. **Read carefully and follow the instruction for use**
2. Do not use reagents and instructions from other kits
3. Bring all reagents to room temperature (+20-25°C) 1 hour before running the test
4. Perform the test at room temperature and keep it constant throughout the execution
5. Do not use kit and reagents after the expiration date
6. Avoid thermal stress: use all reagents for the time strictly necessary for the test and store them at +2-8°C as soon as possible
7. Take out only the strips needed to perform the test, reseal the remaining unused ones in the appropriate bag with desiccant, seal and store at +2-8°C
8. Prepare samples according to instructions: if possible, avoid using lipemic and hemolyzed samples
9. Always use the positive and negative control in each analysis
10. Mix reagents before use
11. Respect the times of each incubation and the order of deposition in the plate
12. Use a new tip for each sample/reagent to be analyzed, avoiding contamination

13. TMB substrate is very sensitive to light and contamination. Do not insert the tip directly into the vial. Handle with care
14. The stop solution is a strong diluted acid. Handle with care
15. All components should be stored at +2-8°C
16. Do not smoke, eat or drink where specimens or kit reagents are being handled

PREPARATION OF REAGENTS

Wash buffer solution:

Dilute one part of the wash buffer supplied into 9 parts of distilled or deionized water (ie. 100 ml of concentrated wash buffer with 900 ml of water). Once diluted, the wash buffer solution must be stored at +2-8°C.

WASHING STEPS

The washing steps may be carried out using an automated plate washer or a multichannel pipetting device suitable for dispensing 300 µl in each well. After the incubation, the washing steps should be carried out according to the following instructions:

- throw out the content of the plate by vigorously turning the plate over to avoid the possible mixture of the content from one well to another
- dispense a volume of 300 µl of diluted wash buffer into each well
- shake the plate gently, avoiding contamination between wells
- turn over the plate and tap the wells vigorously to remove the wash buffer
- repeat the process as many times as is indicated by the kit's instructions
- prior to emptying out the plate's contents during the final washing step, ensure that the next reagent to be added to the plate is ready to use
- the wells should not be left dry for any longer than strictly necessary in order to dispense the reagent of the following step
- after the last washing step, blot the empty wells face down on clean paper towels or absorbent paper to remove residual wash buffer

PREPARATION OF SAMPLES

Feline serum or plasma samples.

Prepare a **1:100** dilution of each sample to be tested using diluent.

For titration we recommend to assay two-fold dilutions from the previous one (1:200; 1:400; etc.).

TEST PROCEDURE

1. Bring all reagents to room temperature 1 hour before use.
2. Add 100 µl of positive control and 100 µl of negative control in two different wells. Add 100 µl of each diluted sample on the remainder wells of the plate.

Delicately shake the plate, seal and incubate at room temperature for 10 minutes.

3. Wash cycle 4 times according to the procedure previously described.
4. Dispense 100 µl of conjugate into each microwell.

Delicately shake the plate, seal and incubate at room temperature for 10 minutes.

5. Repeat the wash cycle 4 times according to the procedure previously described.
6. Add 100 µl of substrate to each well.

Seal the plate and incubate at room temperature for 5 minutes in the dark.

7. Add 100 µl of stop solution to each well.
8. Measure the absorbance value of each well with a spectrophotometer at a wavelength of 450 nm.

READING AND INTERPRETATION OF THE RESULTS

The reading of the absorbance values (OD) must be carried out at a wavelength of 450 nm. If you run the samples or controls in duplicate wells, the OD value for the sample or controls will be the mean of both wells.

Test Validation

The test is to be considered valid when:

Positive Control: $OD > 1$

Negative Control: $OD < 0.25$

CUT-OFF value determination

Positive Cut-off = $OD \text{ negative control} + 0.25$

Negative Cut-off = $OD \text{ negative control} + 0.20$

Interpretation of the results

Positive Samples: samples with an OD higher than the positive cut-off value.

Negative Samples: samples with an OD lower than the negative cut-off value.

Doubtful Samples: samples with an OD between both cut-offs. In these cases, it is suggested to repeat the test after 3-4 weeks. If there is no seconversion and its OD value is not higher than the positive cut-off in the second test, the sample will be considered negative.

AGROLABO

Manufactured by;
Agrolabo SpA

Head Office, Laboratories
and Production Centre
Diagnostic Division
Via Masero 59
10100 Scarmagno (TO)
Italy
Tel +39 0125 731111
E-mail agrolabo@agrolabo.it
www.agrolabo.it